

3M™ Tegaderm™ IV Securement Dressing Designed for the BD Nexiva™ Closed IV Catheter System

Application Instructions for 9525HP



Step 1

Prep the catheter site according to your facility's protocol. Let all prep solutions dry completely. Remove liner. Place dressing, adhesive side up, on sterile side of dressing package.



Step 2

After catheter insertion, stabilize catheter with one hand. With other hand, grasp paper end of dressing, turn it adhesive side down and bend paper end slightly with thumb.



Step 3

Place dressing so that top edge of window lines up with bottom edge of catheter wings.



Step 4

Press down on slit side of dressing. Separate dressing slit at notched corner revealing a J-shaped slit.



Step 5

Pull extension tubing through the slit ensuring that tubing rests at end of J-curve. Overlap edges to close slit and press in place.



Step 6

Apply pressure over entire dressing surface.



Step 7

Slowly remove the frame while smoothing down dressing edge.



Step 8

Smooth the dressing from the center toward the edges, using firm pressure.



Step 9

Remove soft cloth tape from liner and apply to secure extension tubing.



Step 10

Record information on tape label and place it on or near dressing.

Removal Instructions for 9525HP



Step 1

Remove tape strips. Then gently grasp soft cloth on both sides of slit and separate.



Step 2

Slowly peel dressing back over itself while stabilizing catheter and supporting skin.